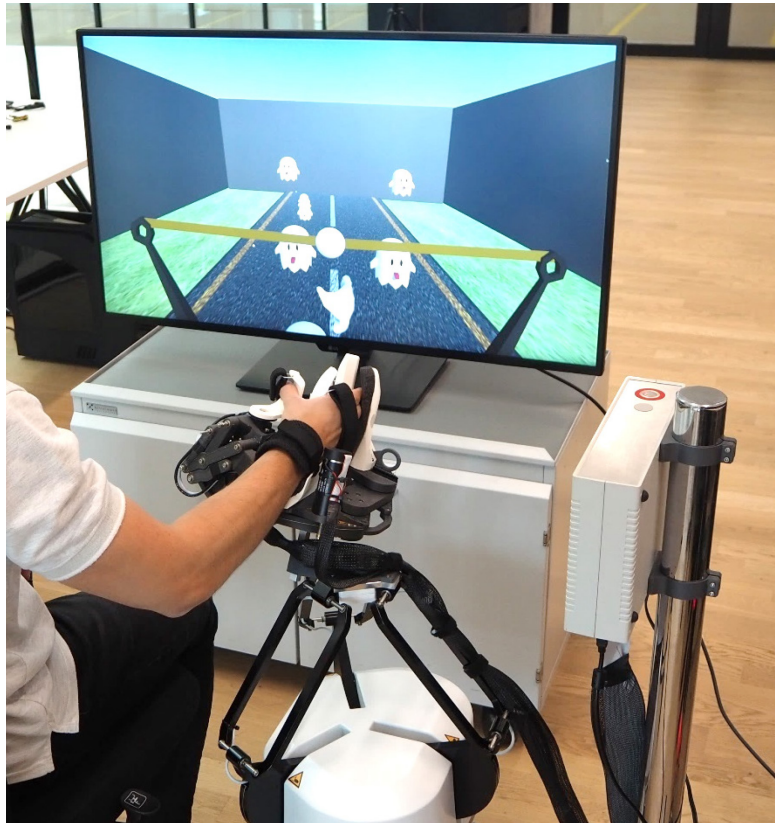
LAMBDA.3+

Version 1.0

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

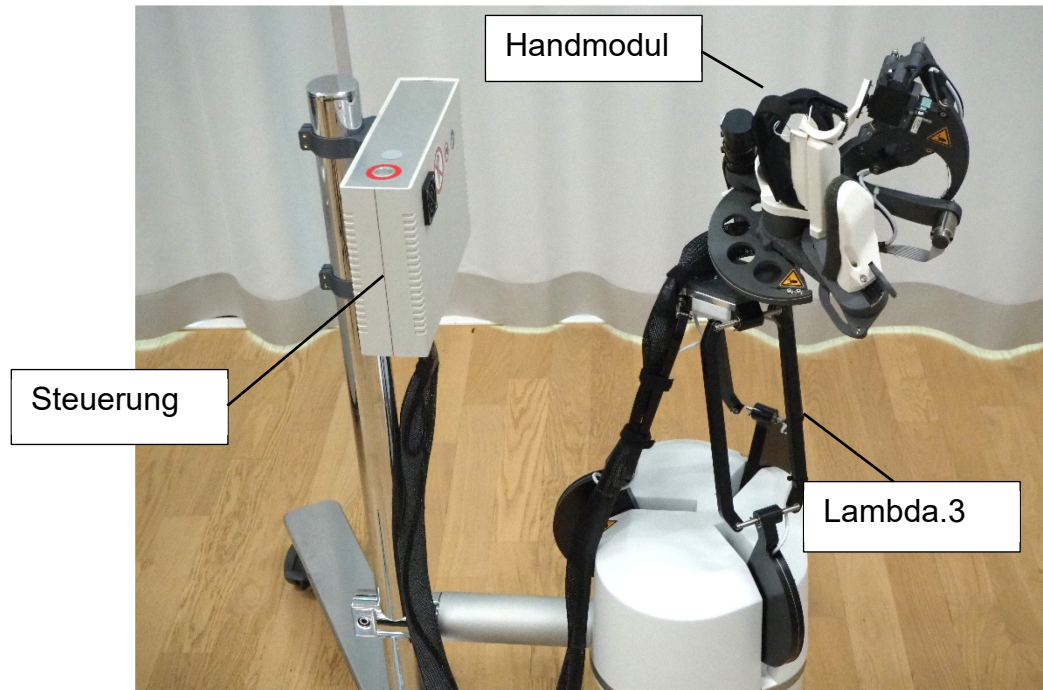
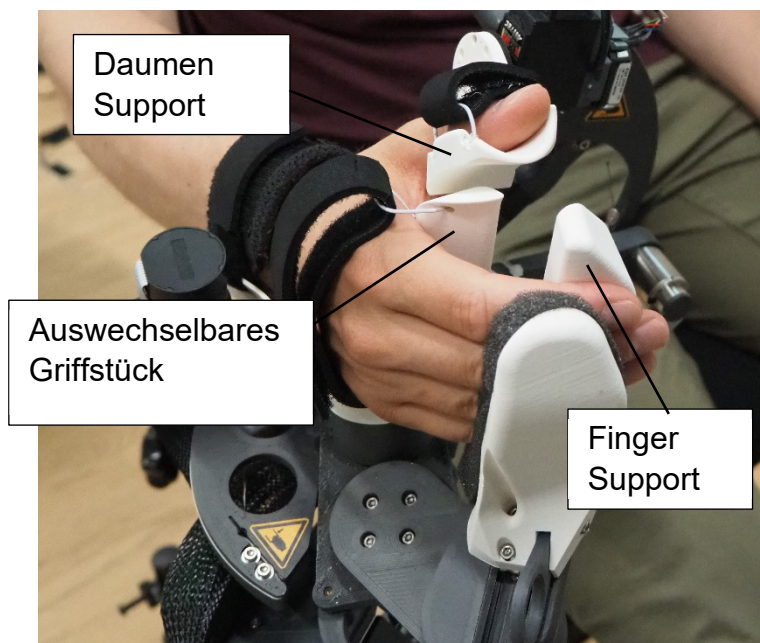
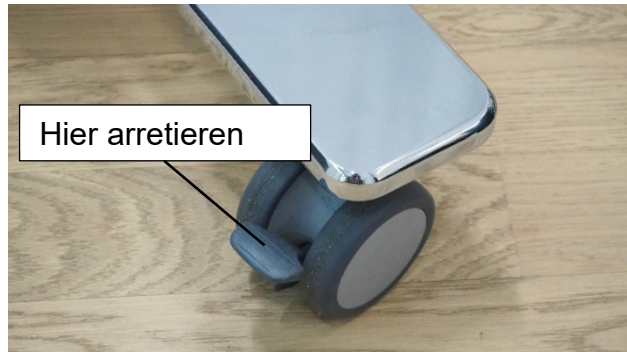
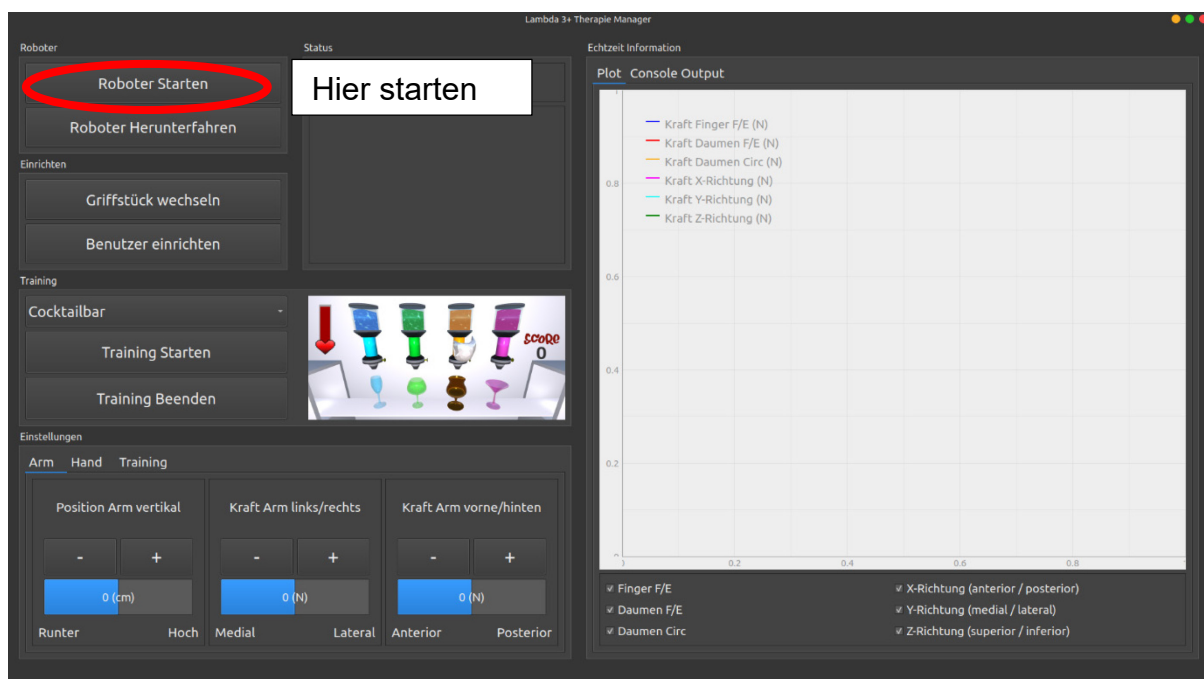
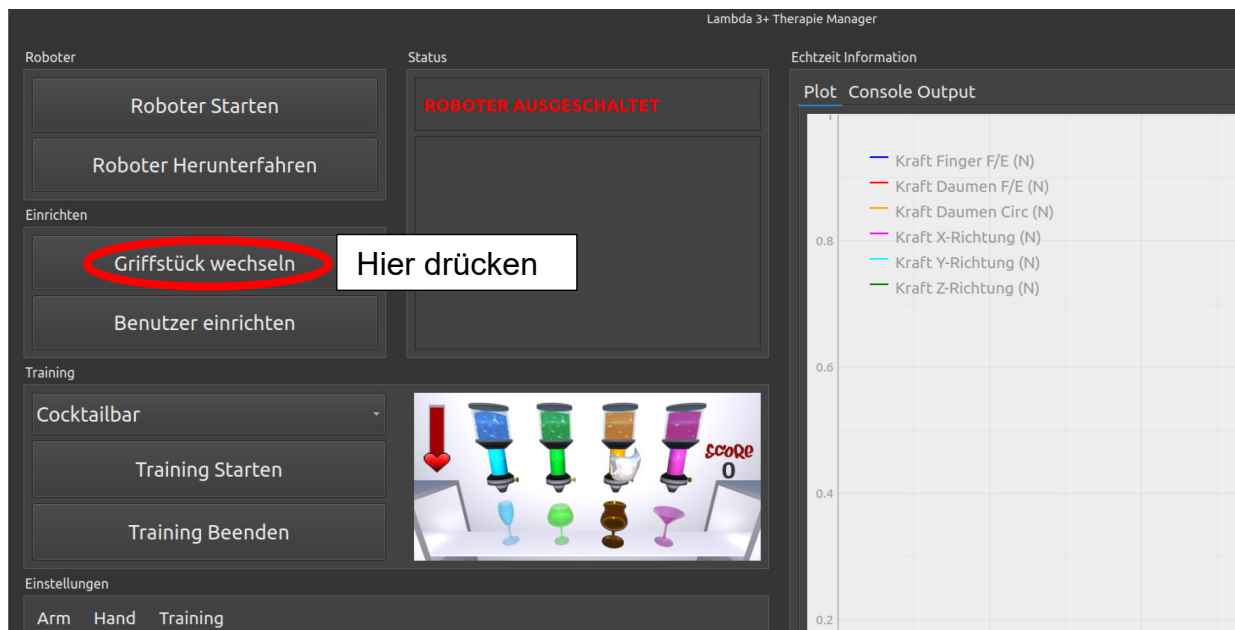
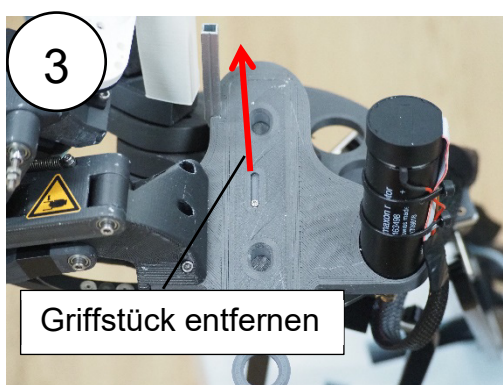
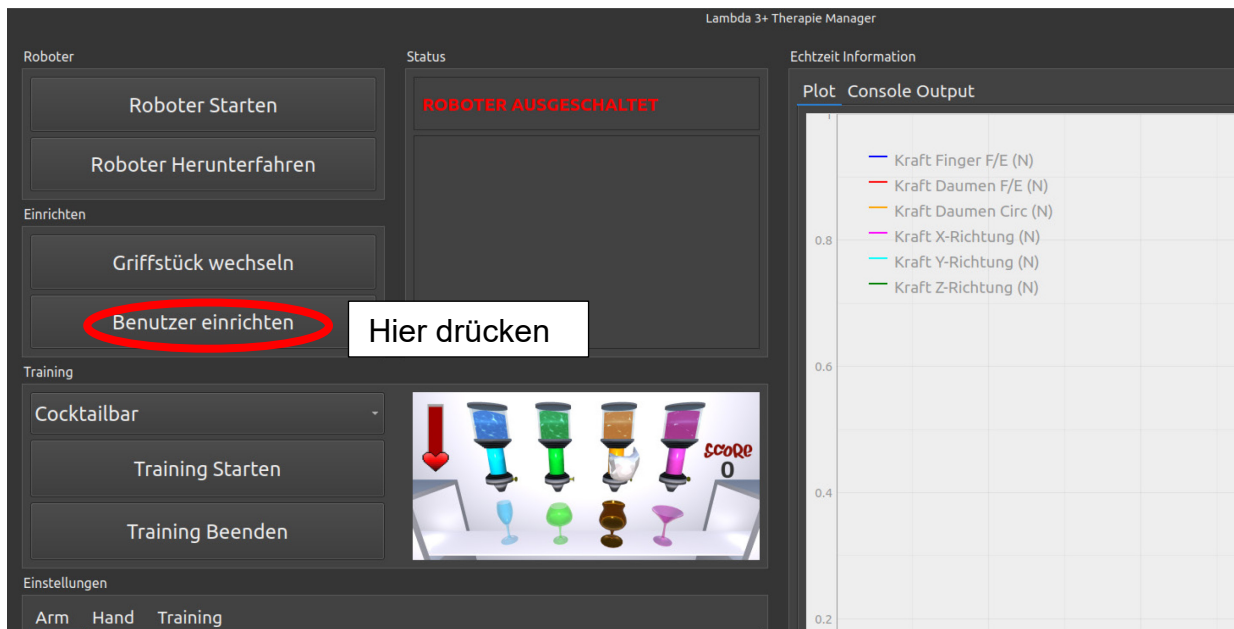
1) Hand fixieren

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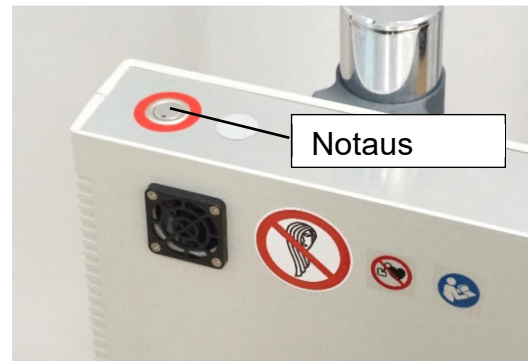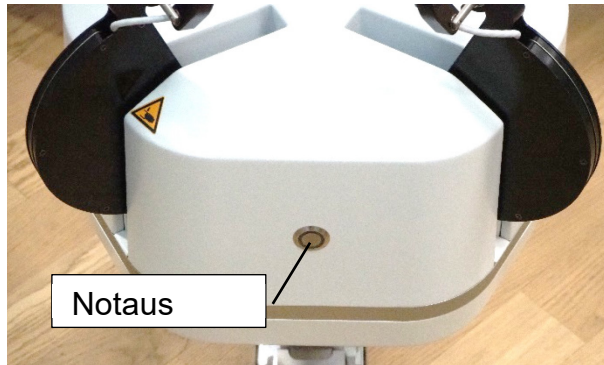1) Notaus

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The screenshot shows the "Lambda 3+ Therapie Manager" software interface. The interface is divided into several sections: "Roboter", "Status", "Einrichten", "Training", and "Einstellungen".

- Roboter:** Contains buttons for "Roboter Starten", "Roboter Herunterfahren", "Griffstück wechseln", and "Benutzer einrichten".
- Status:** Displays "ROBOTER BEREIT" in green.
- Einrichten:** Contains buttons for "Griffstück wechseln" and "Benutzer einrichten".
- Training:** Contains buttons for "Cocktailbar", "Training Starten", and "Training Beenden". The "Cocktailbar" button is circled in red, with a white label "Hier auswählen" pointing to it. The "Training Starten" button is also circled in red, with a white label "Hier starten" pointing to it.
- Einstellungen:** Contains sub-sections for "Arm", "Hand", and "Training". The "Arm" sub-section has buttons for "Position Arm vertikal", "Kraft Arm links/rechts", and "Kraft Arm vorne/hinten". These buttons are circled in red, with a white label "Optional: zusätzliche Einstellungen" pointing to them.
- Echtzeit Information:** Displays a "Plot Console Output" showing real-time data for various force components (Kraft Finger Fx, Kraft Daumen Fx, Kraft Daumen Cy, Kraft X-Richtung, Kraft Y-Richtung, Kraft Z-Richtung) over time.

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